




How to inject with a Pen Needle

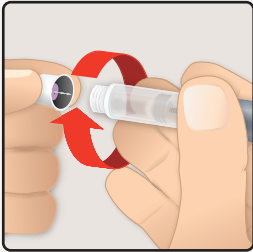


1. Wash hands. Remove seal and push the new needle straight onto the pen. **Do not put the needle on at an angle.** Screw on tight. Remove outer cover.

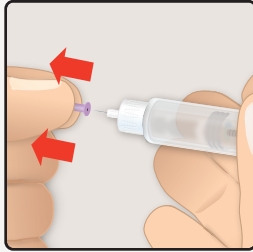


2. Remove needle inner shield.

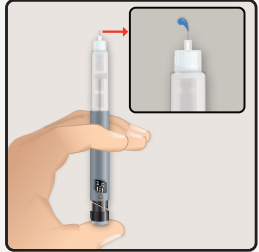

3. Check the flow of medication by dialing 2 units until you see a drop of medication.

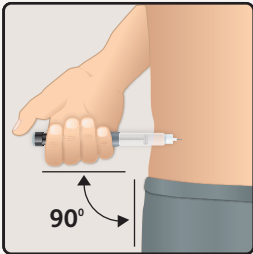

4. Inject straight in.*

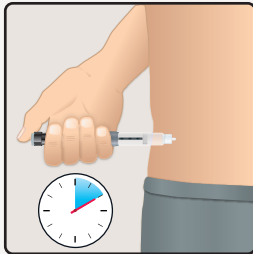

5. Post-injection, count for 10 seconds before removing from skin to help ensure an accurate dose.




6. **Use once** and dispose of properly. **Do not leave pen needle attached causing an open passage for air.**


*Patients ages 2-6, or those who are extremely lean may require a pinch-up.

For additional information, visit bd.com/sharpsdisposal


Practice Healthy Injection Site Rotation

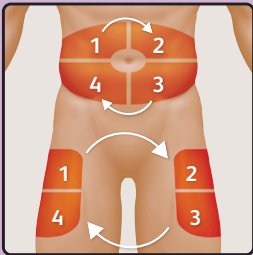


Injection sites:
Front

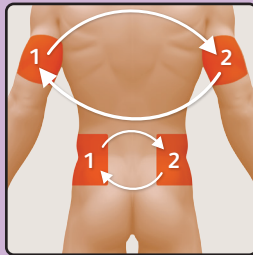

Injection sites:
Back

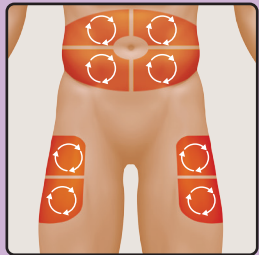


Rotate both between
and within sites

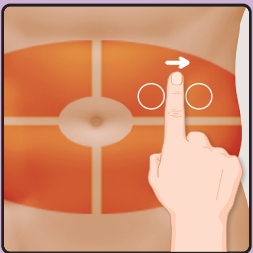


When returning to an injection site, inject at least one finger width away from your last injection.

Replace Your Needle Every Time You Inject

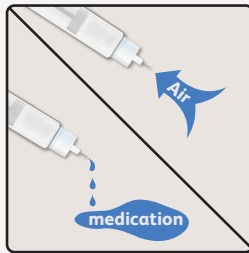

- Do not leave pen needle attached causing an open passage for air.

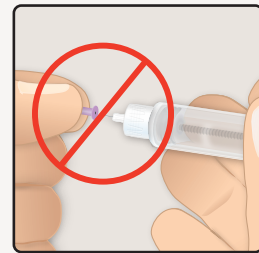

- Do not resheed used needles.